

Care & Information Sheet

- It is recommended that you regularly wash your cover to avoid dirt build-up, which may over time change the colour of the silicone.
- Wash as required with warm mild soapy water and rinse well.
- Clean any accumulated grime or marks with a soft brush or cloth.
- Never use harsh cleaning chemicals or put it in the washing machine.
- Always air or towel dry and avoid putting on hot surfaces.
- Strong marks such as shoe dye or biro ink should be washed off before they are allowed to become ingrained.
- Wearing tight shoes may cause colour and detail of silicone to fade over time.
- Silicone covers designed for use with heels should always be supported by shoes, when wearing the limb; otherwise the top of the foot may eventually crease and buckle, causing severe whitening and discolouration along the creases.
- Take care with shoes with coloured suede on the inside, as this may permanently mark the silicone.
- Take care with blue dye from tight jeans which may stain silicone.
- Sun tan/fake tan lotions may permanently stain the silicone, owing to its semi-permeable nature.
- Avoid contact with sharp objects.
- Great care must be taken when kneeling. Never support your body weight on the knee of your residual limb, or this may cause the socket or prosthetic component to cut into the silicone.
- Nail varnish can be applied to nails but may not be long lasting. Repeated use of acetone (nail varnish remover) may, over time, prove harmful to the silicone.
- Silicone can slip on wet surfaces; therefore care must be taken when wearing the product without footwear.
- If used in chlorinated or salt water the SAFO should be rinsed through thoroughly with fresh water after use.
- If you develop skin reaction to the silicone, please contact your clinician.

Every attempt is made to colour match the cosmetic cover to the closest match possible, slight variations may be apparent under different lighting conditions.

All products are custom made and are allowed to use according to the purpose they have been made for. The custom made products cannot exchange or return for credit. There isn't any liability for material defects due to damage caused by improper use or care.